

Month/Year:

Serial Number:

## ESH Touch Preventive Maintenance Schedule

For more information on correct procedures, read the product inserts contained in every box of gels.

Daily Items, If Used	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	
Prime ESH Touch																																
Rinse sample trays with water																																
Wash ESH Touch																																
<b>Weekly Items</b>																																
Clean/Bleach DI water & Saline bottles																																
<b>Monthly Items</b>																																
Bleach ESH Touch																																
<b>As Needed</b>																																
Replace tubing/pumps																																
Replace fuse																																
Replace/Calibrate probe																																
Backup parameters																																
Replace syringe seal / o-ring																																
Inspect/Clean probe and probe tip																																

Note: Duplicate this page for maintenance copies. Initial each item as required. Refer to Section ten, Maintenance, of the Operator's Manual for details. If you are uncertain about how to perform any step, please contact Helena Laboratories Electronic Customer Service Dept. (1-800-231-5663) for more information.  
Helena Laboratories D7000010A, 12/16